

# Lite Spirit Technical Instructions

Models: 2924, 2925, 2926, 2927



# Lite Spirit | Glazing with Silhouette Vision Sensation® Lenses

Silhouette now offers premium prescription lenses for all Silhouette frames.

Expertly crafted in Austria using our exclusive Panorama® lens designs, Silhouette Vision Sensation® lenses perfectly match the curve of every Silhouette frame.

This results in a complete visual solution that offers the wearer exceptional visual comfort and edge to edge clarity.

For more information visit:

<https://www.silhouette.com/us/us/vision-sensation>

or contact your Silhouette Account Executive



# Lite Spirit | Glazing

Frame is designed with a 5.0 base curve

Other base curves will work and should have V-bevel profile to match 5.0 base curve to maintain frame curve

Recommended lenses are High Index 1.6, 1.67, Polycarbonate and Trivex

Patterns available from Silhouette customer service at 800-223-0180

OMA shape files available at  
<https://portal.silhouette.com>  
[www.silhouettelab.com](http://www.silhouettelab.com)



**Do not heat frame to aid lens insertion!**  
If lenses are large, resize to allow cold insert.

# Lite Spirit | Temple mounting

Lite Spirit temples are attached to frame front using BLS sleeve.

In the event of temple needing replacement, use BLS 58 sleeve to remount temple

Compress with Silhouette mounting plier and turquoise or blue tip





# Lite Spirit | Inclination



1. Open the temple.
2. Support BLS temple fixation firmly with the mounting pliers.
3. Hold end piece firmly and adjust inclination into the required position.

# Lite Spirit | Temple opening

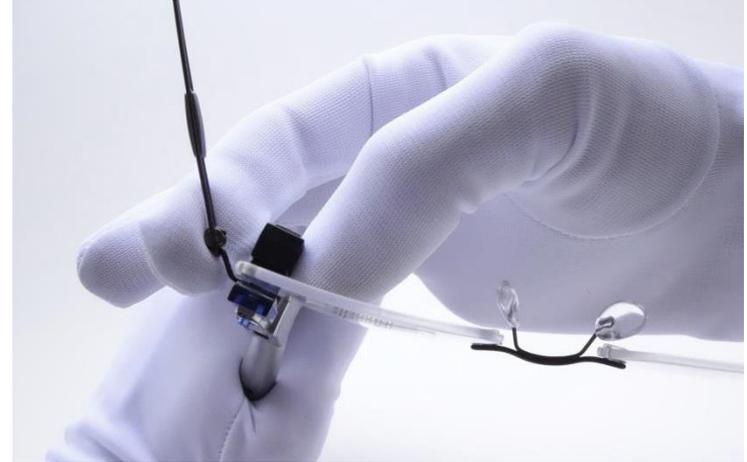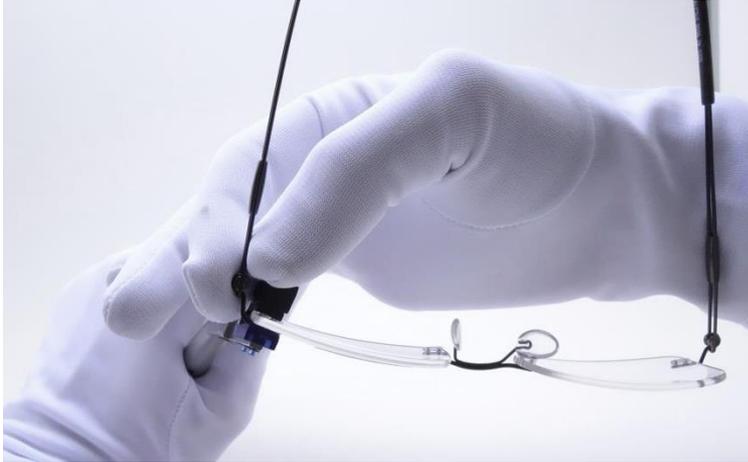

Hold the BLS temple fixation with the mounting pliers and press the temple outward with your thumb or inward with your index finger

# Lite Spirit | Temple curving



Use large curves to adjust  
the temple to fit the head