

Silhouette®

SPX Motion Adjustment Instructions



HOW TO ADJUST BRIDGE:

- Use air heater with small cone

NOTE: do not overheat plastic parts!

- Warm bridge carefully until SPX loses tension (*approx 176° F*)
 - Form it at room temperature
- Keep position until bridge has cooled down



HOW TO ADJUST TRIM:

- Fix trim with assembly pliers (pic 1)
- Use air heater with small cone


NOTE: do not overheat plastic parts or ophthalmic lenses!

- Warm up carefully until SPX loses tension (*approx 176° F*)
- Form manually when SPX component becomes soft (pic 2)
 - Keep in position until trim has cooled down



HOW TO DETACH TEMPLE:

- Position round stick (1.5-2 mm diameter) close to the hinge
- Fold the temple and press together with thumb

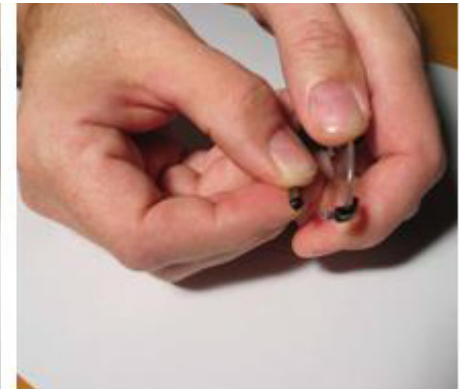


Silhouette®

SPX Motion Adjustment Instructions (continued)



HOW TO ADJUST TEMPLE:

- Warm up temple carefully (pic 1)
- Gently bend curve (pic 2)

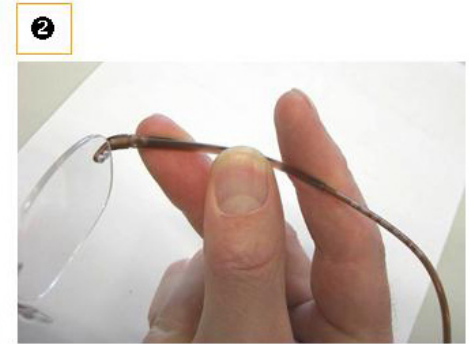

HOW TO ADJUST THE INCLINATION OF THE TEMPLE:

- Disassemble temple with round stick:
(1.5-2mm diameter (pic 1))
- Fix trim with assembly pliers

