

Dynamics Colorwave | Technical Instructions




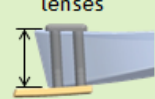
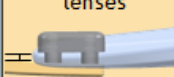



Glazing I Dynamics Colorwave

Frame is designed with 5.0 base curve. Other curves can be used and will require bridge adjustment for proper lens wrap.

Use Silhouette mounting plier with blue tip and BLS 58/98/97 for glazing

Recommended lenses are polycarbonate, Trivex, 1.60 and 1.67 high index



BLS 58 STANDARD	BLS 98 „long“	BLS 97 „short“
Standard Glazing 	for very thick lenses 	for very thin lenses 
lens size 2,0 - 6,5 mm	6 - 9,5 mm	1,3 - 3 mm
		
∅ 1,4 mm	∅ 1,4 mm	∅ 1,4 mm

Glazing I Dynamics Colorwave

Lens shape BG special instructions for browbar

Glaze shape BG with 19mm bridge only
Browbar attaches with BLS 44 sleeve
Compress single sleeve with green plier tips



BLS 44
„single“

for single drill
holes




2,0 - 6,5 mm



1,4 mm




Inclination | Dynamics Colorwave

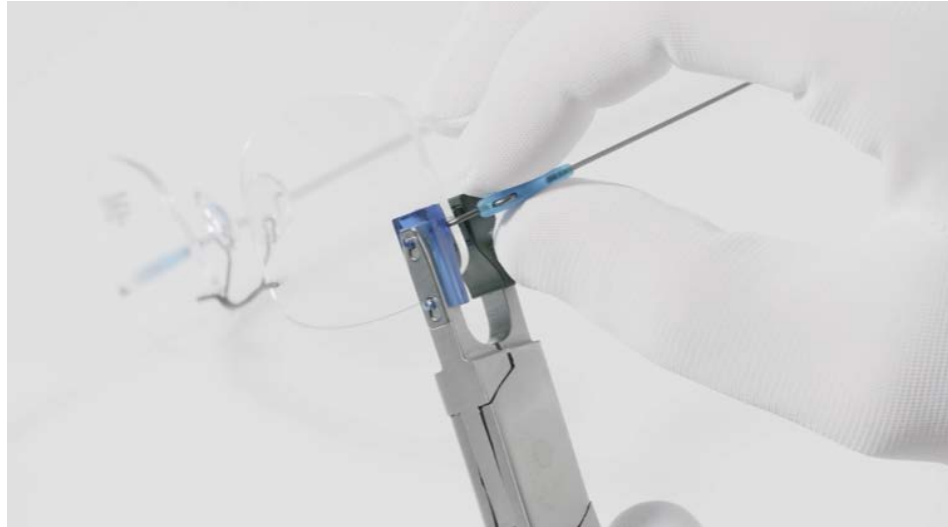


1. Open the temple.
2. Hold lens mounting firmly with the mounting pliers and blue tip.
3. Adjust inclination at endpiece into the required position.

Temple shortening | Dynamics Colorwave

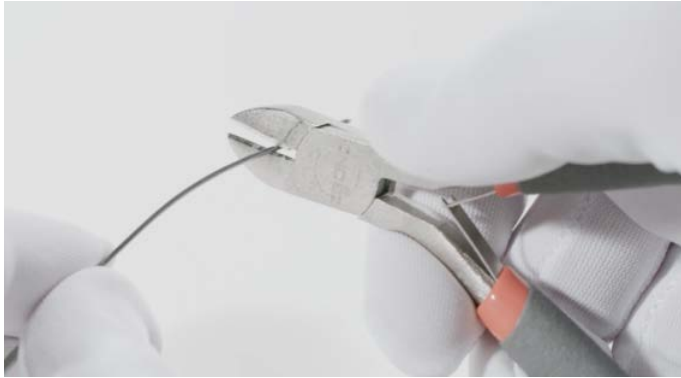

1. Remove the temple end.
2. Shorten titanium wire with metal-diagonal cutter.
3. File down sharp edges and burrs.

Temple shortening | Dynamics Colorwave

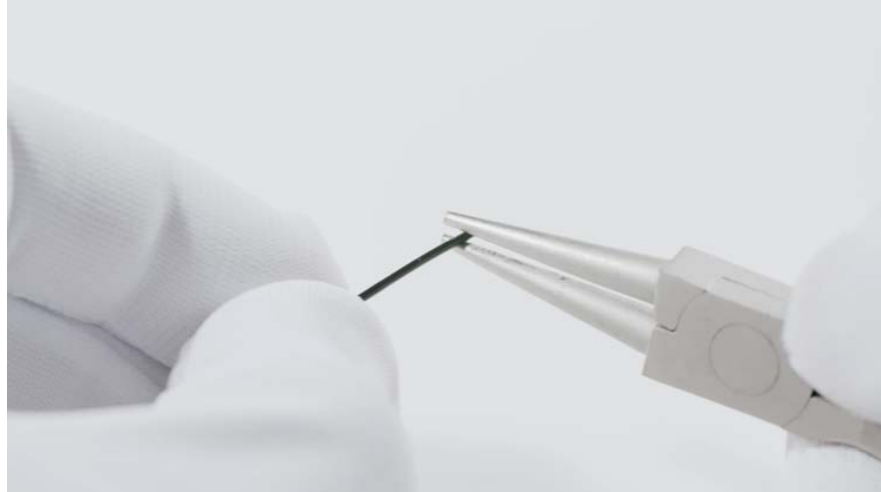

1. Squeeze titanium wire with round pliers at the very end.
2. The notch created at the end of the metal core will prevent the temple end from slipping off.

Temple removal | Dynamics Colorwave



1. Hold lens mounting firmly with your fingers
2. With the other hand grasp the partially closed temple directly behind the hinge and push it in the direction of the lens to remove
3. Turn the temple 180° to the front and slide it off the end piece.

Temple mounting | Dynamics Colorwave



1. Hold lens mounting firmly with your fingers
2. With temple upside down insert the front hole of the temple hinge onto the endpiece.
3. Slide the temple all the way onto the endpiece so it is next to the lens.
4. Rotate the temple 180° so temple tip is facing down.
5. Snap hinge into endpiece.

Temple opening enlarge | Dynamics Colorwave



1. Open the temple
2. Hold lens fixation firmly with mounting pliers
3. Press the temple outward at the hinge area with your thumb

Temple opening reduce | Dynamics Colorwave



1. Open the temple.
2. Hold lens fixation firmly with mounting pliers
3. Press the temple inward at the hinge area with your finger

Temple adjustment | Dynamics Colorwave



Adjust the temple to the ear shape with large curves.
During temple-end adjusting do not put pressure on the temple tip.

Silhouette®

AUTHENTIC EYEWEAR. SINCE 1964.