

Infinity View Technical Instructions

Models 1600, 2937, 2938, 2939
V-Bevel Edging



Infinity View | Glazing with Silhouette Vision Sensation® Lenses

Silhouette now offers premium prescription lenses for all Silhouette frames.

Expertly crafted in Austria using our exclusive Panorama® lens designs, Silhouette Vision Sensation® lenses perfectly match the curve of every Silhouette frame.

This results in a 100% Silhouette eyewear solution that offers the wearer exceptional visual comfort and edge to edge clarity.

For more information visit:

<https://www.silhouette.com/us/us/vision-sensation> or contact your Silhouette Account Executive



Infinity View | Temple Mounting

Mount temples on frame front using BLS 58 sleeve and Silhouette mounting plier with turquoise or blue tip.



Infinity View | Glazing

Frame is designed with a 5.0 base curve, however other base curves will work

Lenses with base curves flatter than 5.0 should have V-bevel cut to match 5.0 base curve

Recommended lenses are High Index 1.6, 1.67, Polycarbonate and Trivex

OMA shape files available at

<https://portal.silhouette.com>

www.silhouettelab.com

Infinity View | Cold Glazing


Cold insert lenses in all Infinity View frames!

Do not heat the frame under any circumstance!

Lenses that are too large will need to be resized!

Always insert lenses from the back of the frame!

Infinity View | Lens Insertion V-Bevel Fullrim

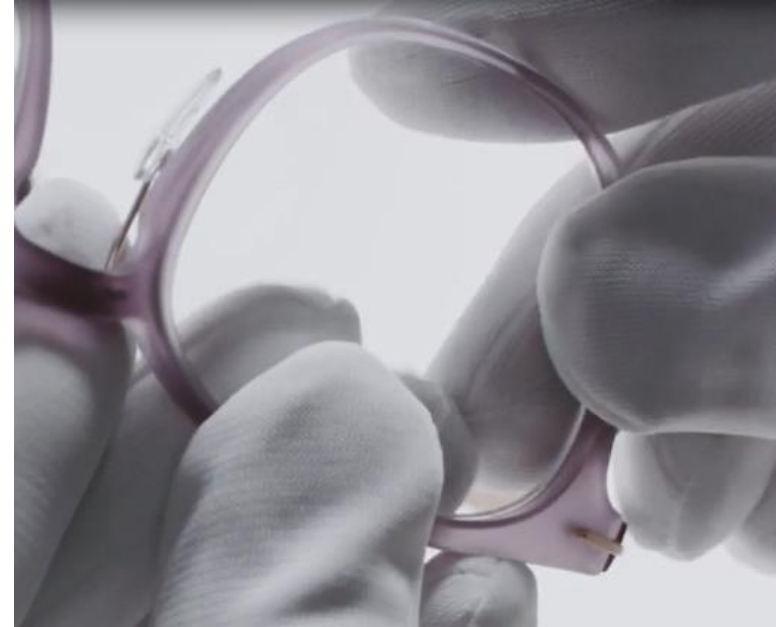

1. Insert the lens from the rear, starting with the nasal corner.
2. Continue inserting the lens along the top to the temporal corner.
3. Continue around the lens circumference until the lens snaps into place.

Infinity View | Lens Removal





1. At the bottom of the frame, partially push frame back and separate the frame from the lens
2. Push lens out of frame toward the front

Inclination | Infinity View




1. Open the temple.
2. Support the temple mount with the Silhouette mounting pliers.
3. Incline the temple at the endpiece in the desired direction.

Infinity View | Temple Opening Increase

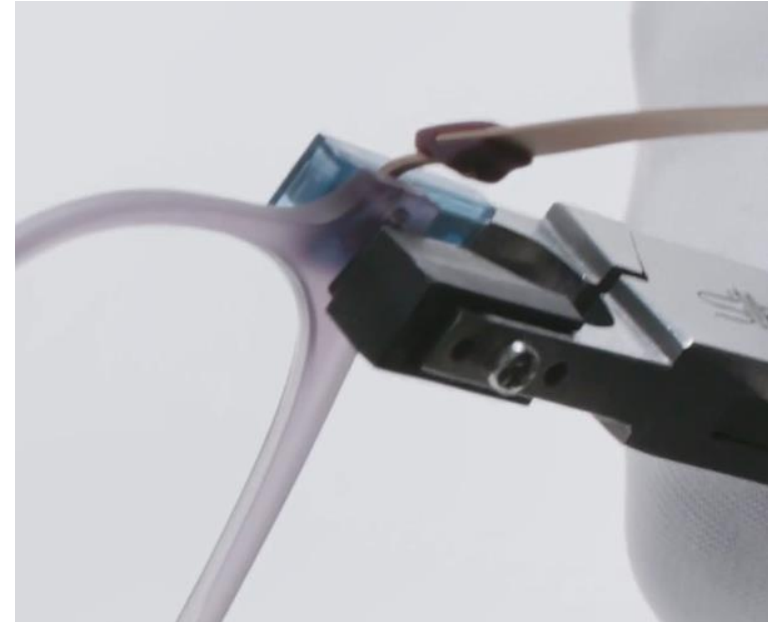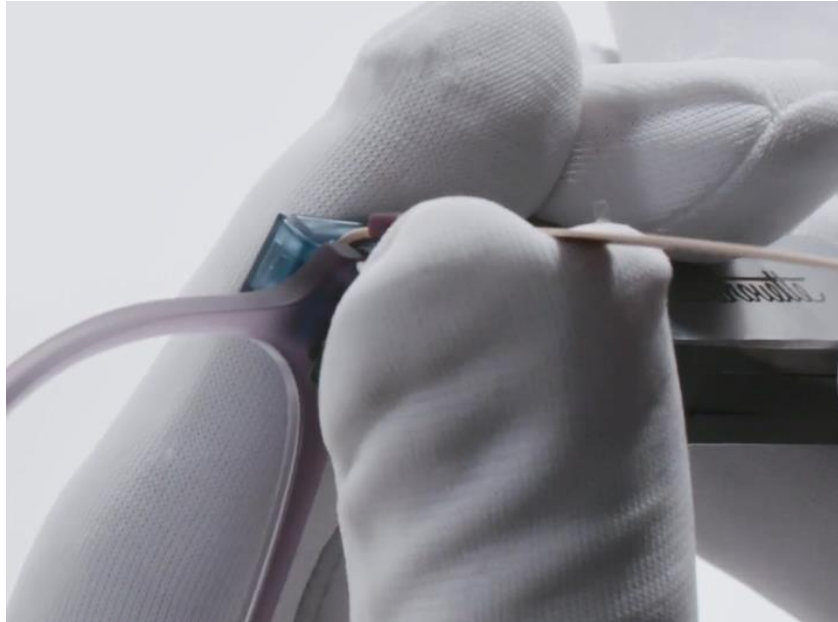

1. Open the temple.
2. Support the temple mounting with Silhouette mounting pliers.
3. Press the temple out at the endpiece with thumb or index finger.

Infinity View | Temple Opening Decrease

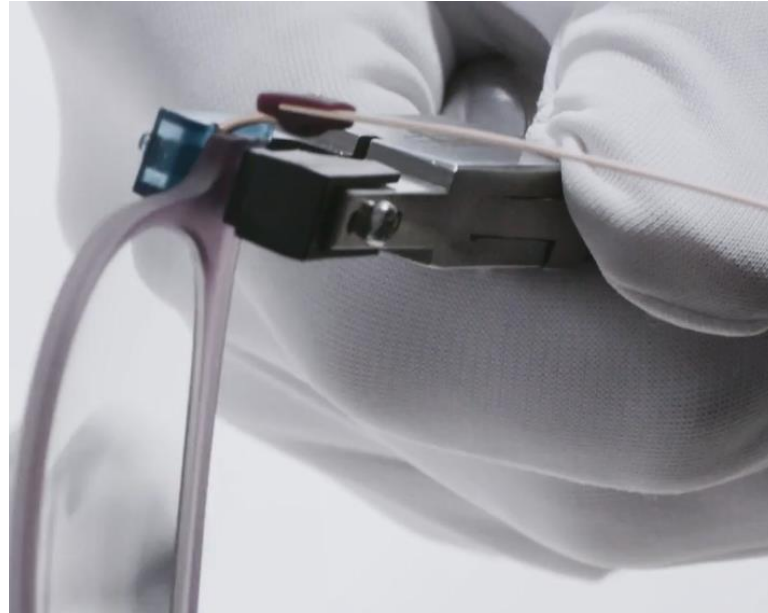

1. Open the temple.
2. Support the temple mounting with Silhouette mounting pliers.
3. Press the temple in at the endpiece with thumb or index finger..

Infinity View | Temple Bend Adjustment




1. Adjust the temple to the ear shape with large curves.
2. Temple can be heated to aid adjustment.
3. During temple-end adjusting do not put pressure on the temple tip.

Infinity View | Temple Shortening - Glued



1. Straighten the end of the temple.
2. Break the adhesive bond by squeezing with with double-jaw padded pliers.
3. Carefully pull off the temple end.

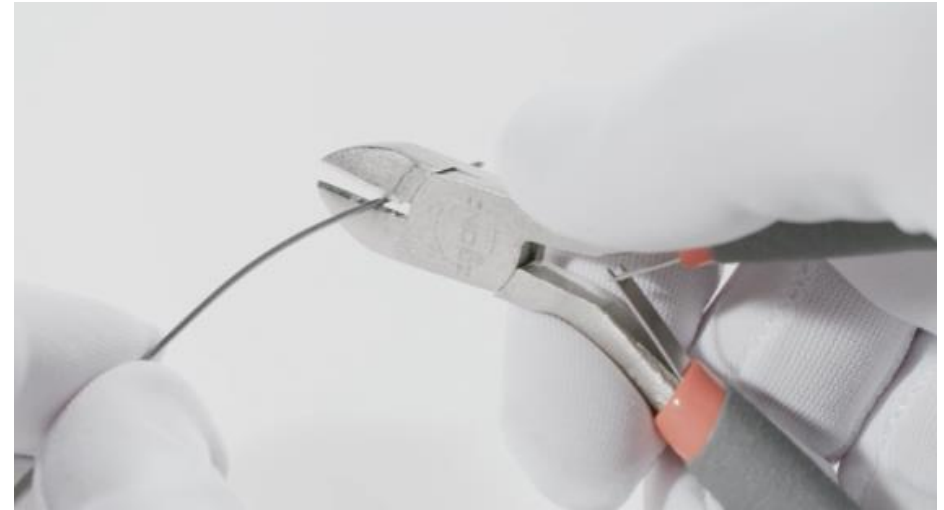

4. Shorten titanium wire with metal-diagonal cutter.
5. File down sharp edges and burrs.

Infinity View | Temple Shortening - Glued




6. Gently remove remaining glue.




7. Cut plastic temple end to required length with sharp blade in the original angle.

Infinity View | Temple Shortening - Glued

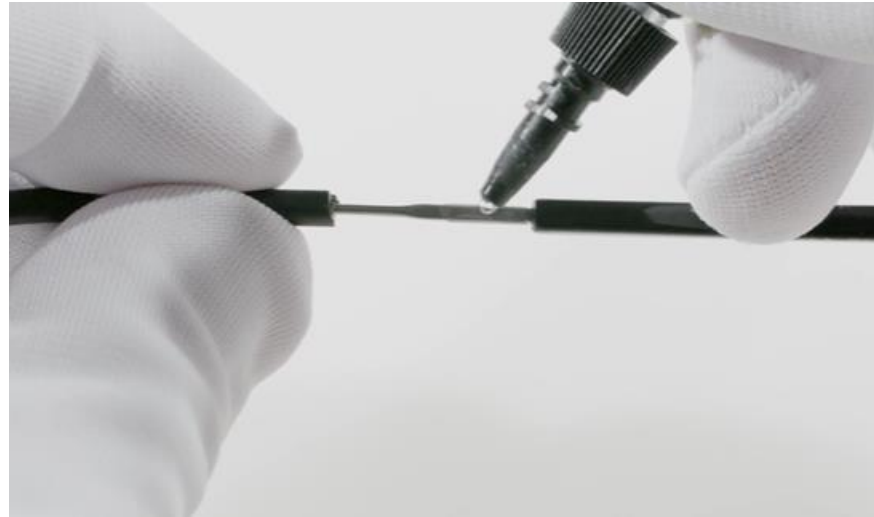


8. Reinstall temple cover.
9. Apply a small amount of glue on the inside of the titanium wire as shown.
10. Slide the temple cover fully onto the titanium wire, wipe off excess glue.

Infinity View | Temple Removal

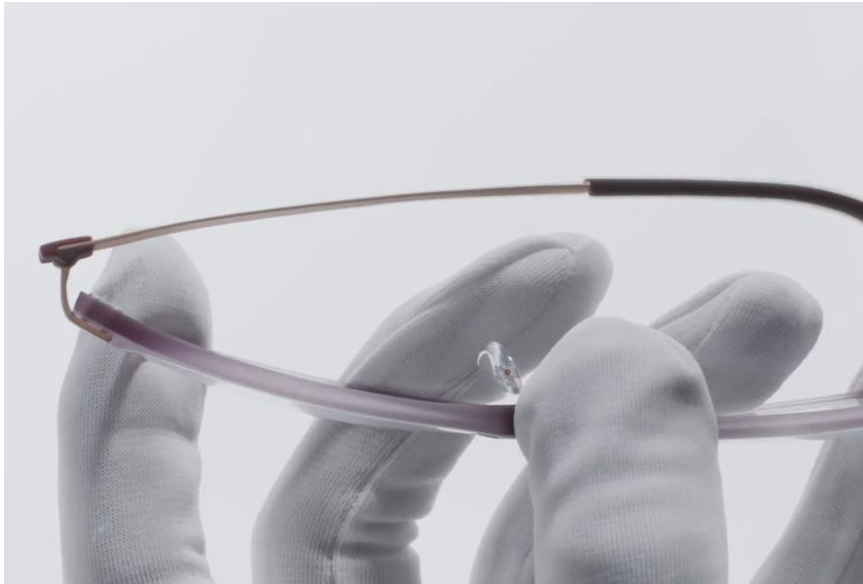

1. Close the temple and hold the lens mount with your thumb and index finger.
2. Using other hand, pull the temple off from the endpiece in direction shown by red arrow.

Infinity View | Temple Install

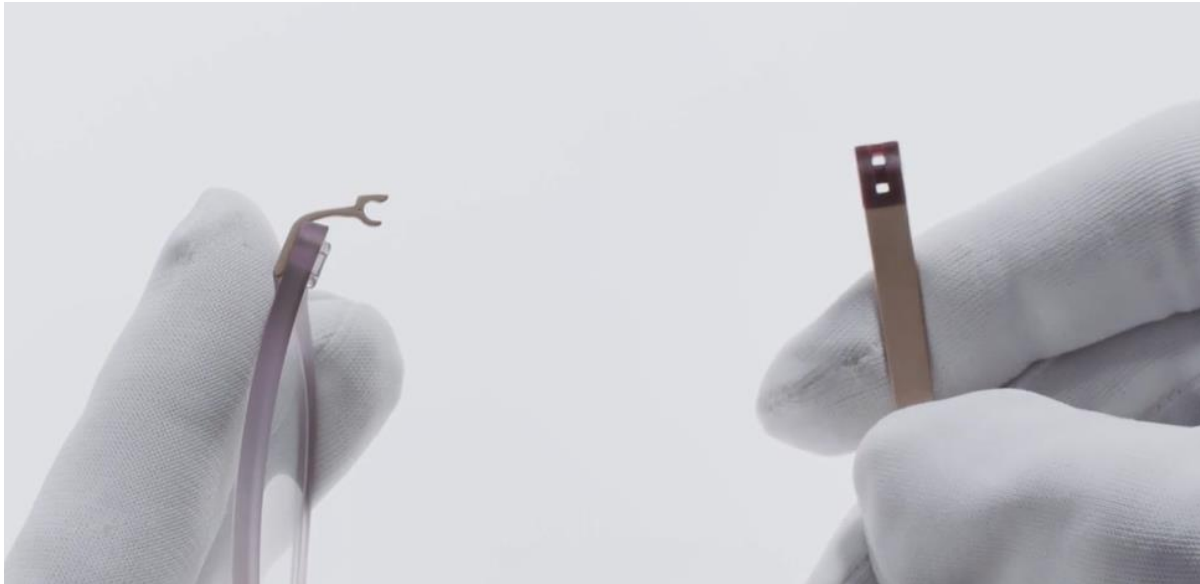

1. Support the frame front as shown.
2. With the temple in the closed position, align the temple to the hinge elements on the endpiece with the temple slots
3. Press temple onto the endpiece

Infinity View | Teflon Lubricant

After cleaning the frame in an ultrasonic bath, relubricate the hinge with Teflon.



Additional information | Infinity View

Cleaning in an ultrasonic bath or with ethanol is possible.

Avoid high temperatures such as the dashboard in the car or the sauna.

Always use the supplied case for storage.



Silhouette®

ICONIC EYEWEAR MADE IN AUSTRIA. SINCE 1964.