

Sivista Technical Instructions Model 5553



Sivista | Glazing with Silhouette Vision Sensation® Lenses

Silhouette now offers premium prescription lenses for all Silhouette frames.

Expertly crafted in Austria using our exclusive Panorama® lens designs, Silhouette Vision Sensation® lenses perfectly match the curve of every Silhouette frame.

This results in a 100% Silhouette eyewear solution that offers the wearer exceptional visual comfort and edge to edge clarity.

For more information visit:


<https://www.silhouette.com/us/us/vision-sensation>

or contact your Silhouette Account Executive



Sivista | Glazing

Mount lenses using BLS 58 sleeve and Silhouette mounting plier with turquoise or blue tip.

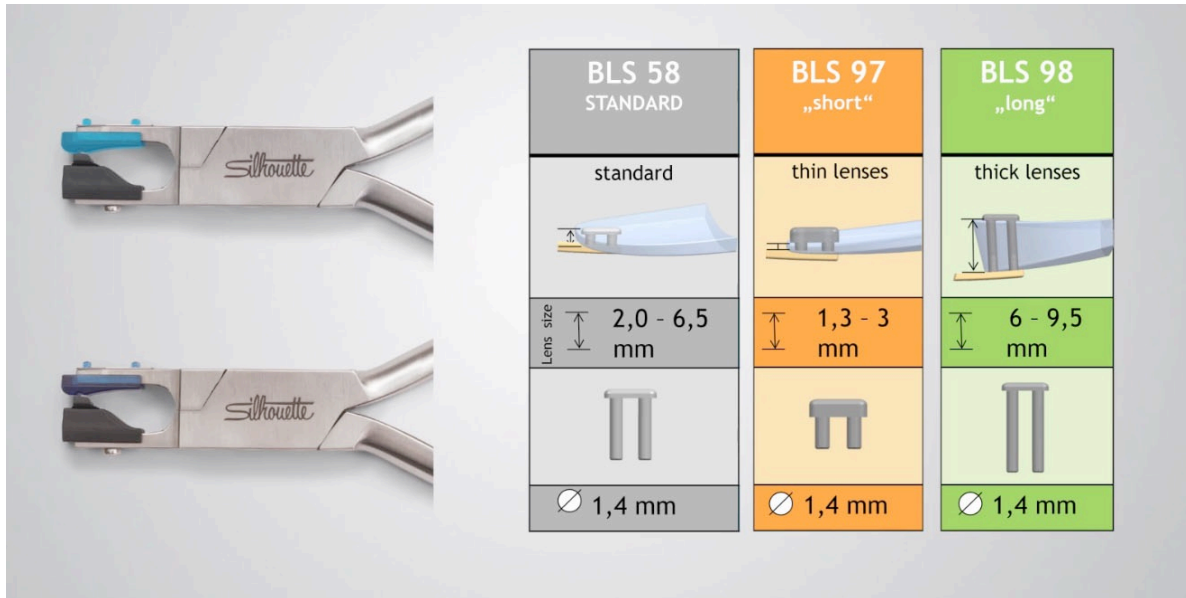

Sivista | Glazing

Frame is designed with a 5.0 base curve

Other base curves will work and will require bridge adjustment

Recommended lenses are High Index 1.6, 1.67, Polycarbonate and Trivex

Patterns available from Silhouette customer service at 800-223-0180


OMA shape files available at

<https://portal.silhouette.com>

www.silhouettelab.com





Sivista | Inclination

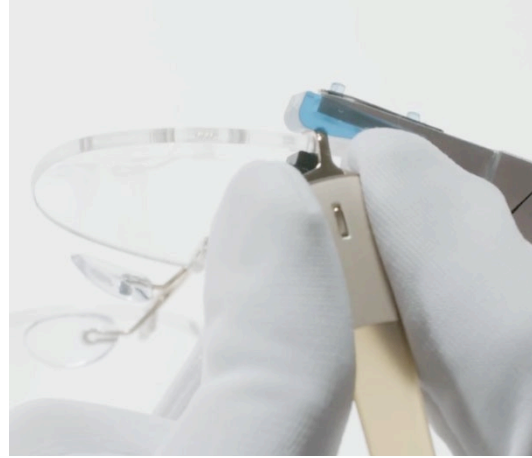



1. Open the temple.
2. Support lens mounting firmly with the mounting pliers.
3. Hold temple end firmly and adjust inclination into the required position.

Sivista | Temple opening increase

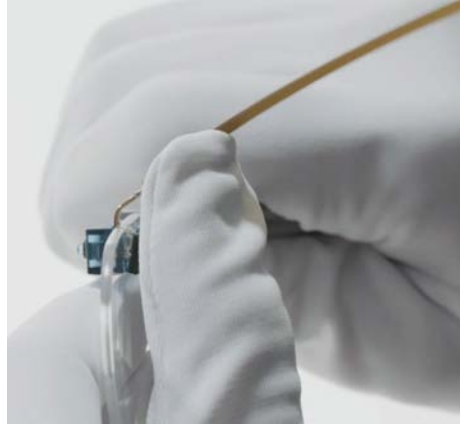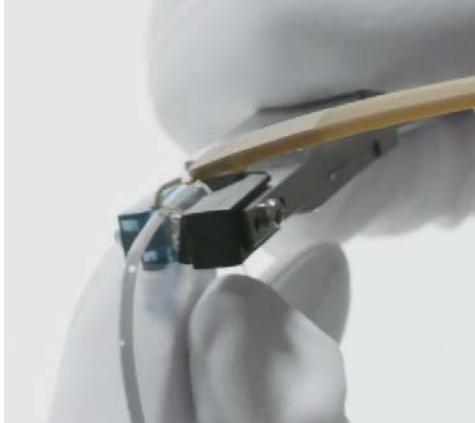


1. Open the temple.
2. Support lens fixation firmly with mounting pliers.
3. Press the temple outward at hinge with thumb.

Sivista | Temple opening decrease

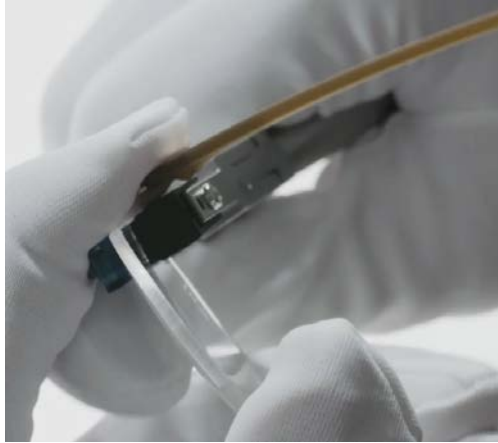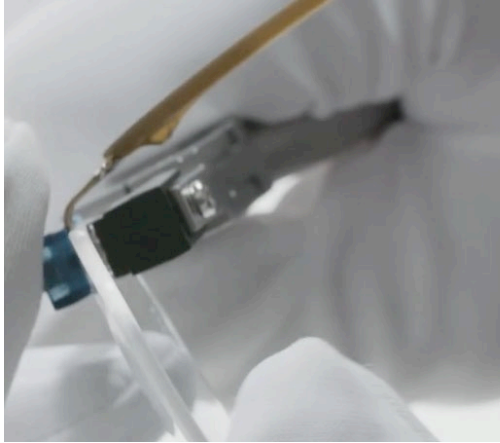

1. Open the temple.
2. Support lens fixation firmly with mounting pliers.
3. Press the temple inward at hinge with index finger.

Sivista | Temple Bend

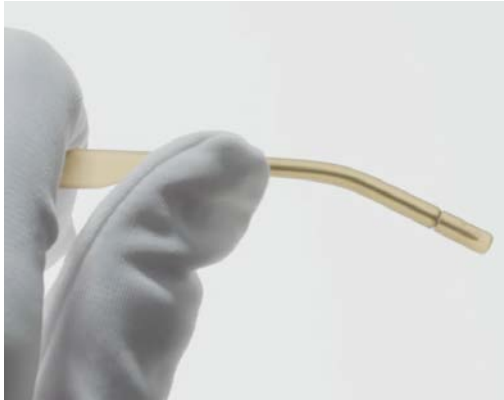

1. Adjust the temple to the ear shape with large curves.
2. Temple can be heated to aid in adjustment.
3. During temple-end adjusting do not put pressure on the temple tip.

Sivista | Temple removal



1. Close hinge and support lens fixation as shown
2. Pull temple from metal hinge in straight up movement as shown by arrow

Sivista | Temple mounting



1. Support lens fixation as shown
2. Align temple openings with metal hinge piece
3. Press temple onto metal hinge piece

Sivista | Teflon Lubricant



After cleaning frame in ultrasonic bath or if temple movement is not smooth, lubricate hinge with a small amount of Teflon lubricant.