

General glazing information:

The distance between the peak of the bevel and the rear surface of the lens in the nasal area should not exceed 5.0 mm so as not to impair the function of the adjustable nose pad (see figures 1+2)

In higher minus-power lenses:

- temple contact with the lens is possible in the temporal area
- the closure of the temple may be obstructed in the temporal area

A lens former is available

Options for adjusting the eyewear to individual wearers:

Inclination


Inclinable metal hinge (note: only use parallel action pliers with plastic jaws) (see figure 3)

Temples

- Marks on the inside of the temples (5 mm spacing) make it easier to shorten the temples (see figures 4-9)
- Bond before reattaching the temples. Sparingly apply super glue gel (cyanoacrylate) such as Loctite 454. Follow security instructions of glue manufacturer (see figure 9)
- The temples can be adjusted to the shape of the head and ears (see figures 10-11)
- Temples are available as spare parts (150 mm)

Ergo-Grip Nose™

- Variable-width, adjustable, soft nose pad (2 positions – see figures 12-13)
- Traction Grip™ technology (no pressure and non-slip) (see figure 14)

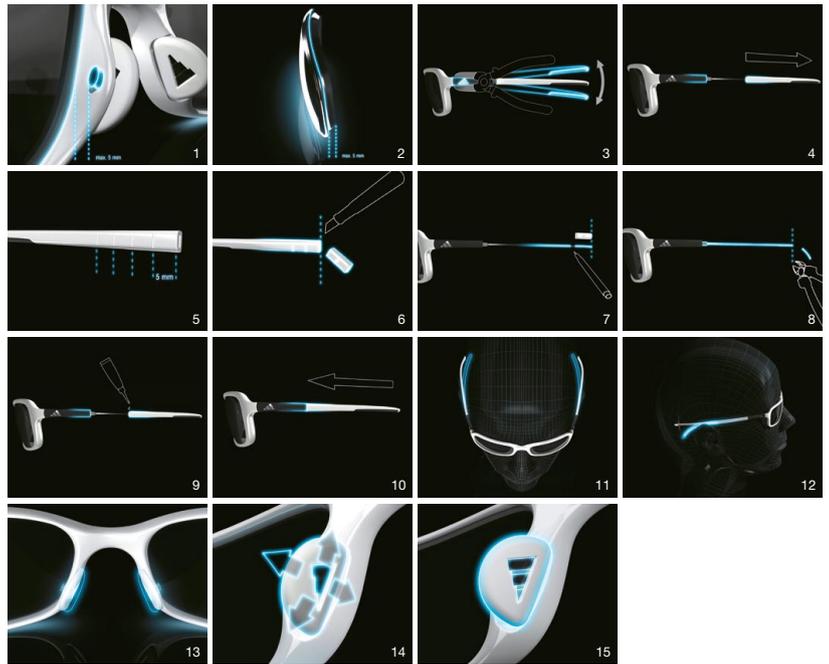

support your vision

Silhouette International Schmied AG

Subsidiary of Silhouette International,
adidas Global Licensee
260 Cannon Street
Green Island, NY 12183
Tel. 1-800-626-8684, Fax 1-518-270-3199
adidas@us.silhouette.com